



DualTec™ Kit

Complete implantation system

The DualTec™ IOL Injector is a single use lens injector for the implantation of one- and three piece acrylic and polysiloxane IOLs. The DualTec™ Kit features complete sets of a DualTec™ Injectors and Locking Wing System™ cartridges required to implant your IOL of choice.



Main Features & Benefits




- ✓ **2.2 mm incision** (*one piece acrylic IOLs*)
2.8 mm incision (*three piece acrylic IOLs*)
3.2 mm incision (*three piece silicone IOLs*)
- ✓ **PUSH or TWIST**
PUSH for one piece IOLs,
TWIST for three piece IOLs
- ✓ **Single use**
Safe, hygienic, constant quality
- ✓ **Cartridge**
Convenient cartridge loading




DualTec™ Kit Loading Instructions


>> Load

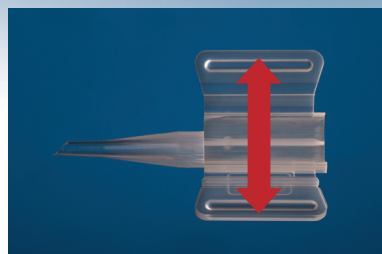

1. Open cartridge;

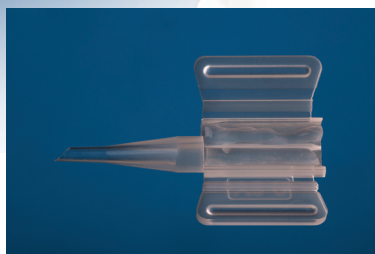


2. Fill cartridge with viscoelastic solution;

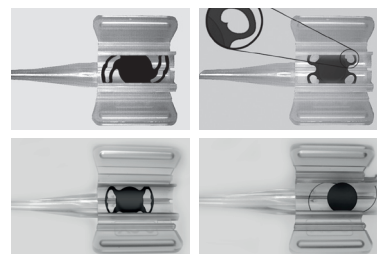

3. Push the lens with the haptics under the rim;




4. Close cartridge until a 'click' is heard;

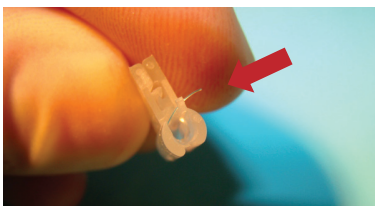


4a. For three-piece IOLs the rear haptic should stick out of the cartridge;


>> Place

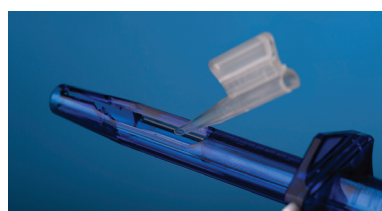

5. Place cartridge into DualTec™ injector;

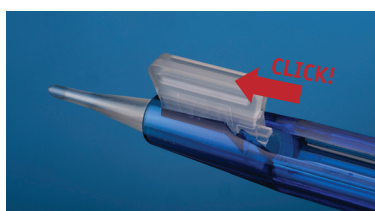


6. Move cartridge forward until a 'click' is heard;

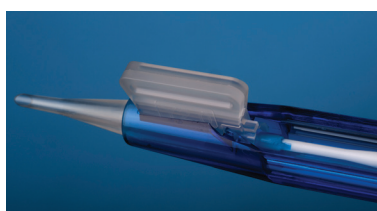


7. Move plunger forward;

>> Insert

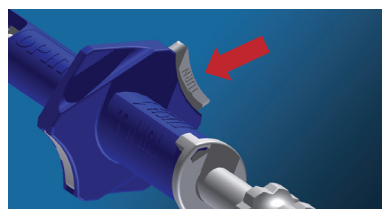


8. Choose PUSH or TWIST using the white switch:

PUSH for one piece IOLs,
TWIST for three piece IOLs;

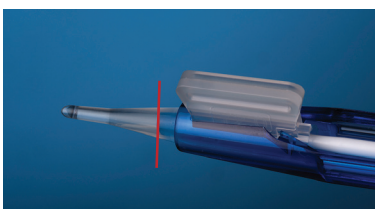

9. **One piece:** Move plunger forward into compression section;

Three piece: Move plunger forward into compression section. Retract plunger and move forward again into compression section;

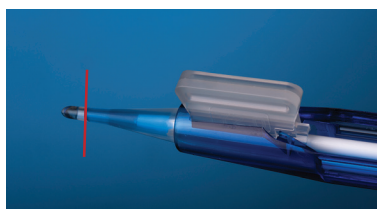

10. Introduce tip of cartridge through the incision and move plunger forward until IOL is out of the cartridge.

Packaging

Injector + cartridge (in blister, box of 10)

- OD 655 DualTec™ Kit 3P Acrylic 2.8
- OD 665 DualTec™ Kit 1P Acrylic 2.0
- OD 640 DualTec™ Kit 3P Silicone 3.0

Cartridge (in blister, box of 10)

- OD 502 LWS™ Cartridge 3P Acrylic 2.8
- OD 522 LWS™ Cartridge 1P Acrylic 2.0
- OD 401 LWS™ Cartridge 3P Silicone 3.0



www.ophtec.com

facebook.com/ophtec

youtube.com/ophtecbv

twitter.com/ophtecint

in

OPHTEC
focus on perfection

OPHTEC BV P.O. Box 398 | 9700 AJ Groningen | Schweitzerlaan 15 | 9728 NR Groningen | The Netherlands | T: +31 50 5251944 | F: +31 50 5254386
E: info@ophtec.com | Information on this document is subject to modification | ©2015 OPHTEC BV | L72-MD47-2