

Ready to use packaging

→ Procedure


ophtec



1a. Pull lid lightly obliquely away from the container.



1b. Remove lid completely, **place container on stable surface!**

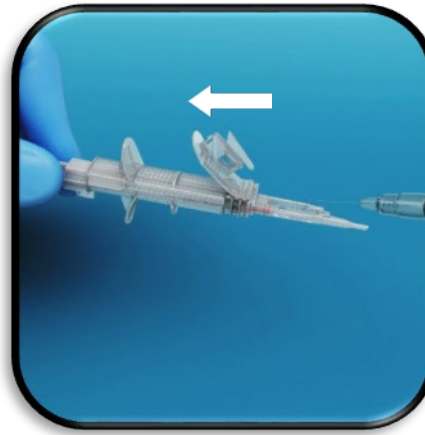


2. Add viscoelastic to cartridge body, **move folding unit backwards!!!**



3a. Place injector vertically in to container to connect lens holder. Push until the **“click”!**



3b. Remove injector from container.

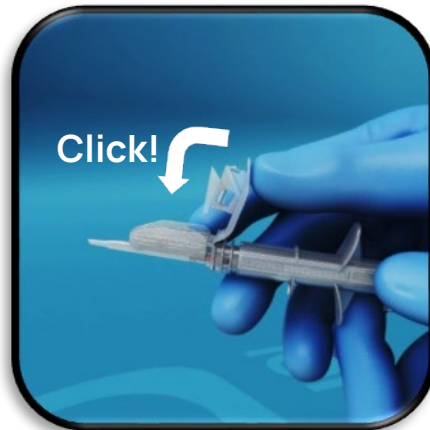

4. Close folding element **in one movement** until “click”.



5. Remove lock.



6. “Ready to use”. Push lens out of cartridge.

This document is for training purposes only. Always consult the IFU of the IOLMATIC Injector System and the concerning IOL for complete instructions.


Pitfalls



1a. Torn lid.
Action: Remove lid completely.



1b. Lid not completely removed.
Action: Remove lid completely.

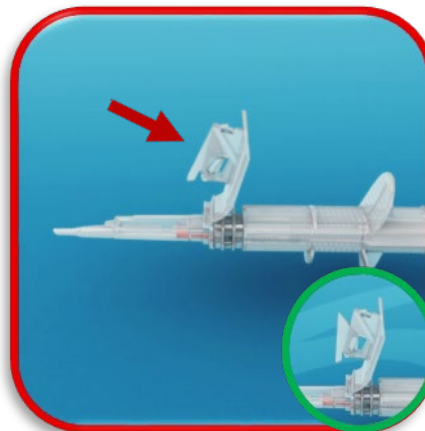

2. Incorrect position folder.
Action: Replace injector.



3a. Lens holder misaligned.
Action: Reassemble lens holder.




3b. No "click".
Action: Push lens holder until "click".



4. No "click".
Action: Replace injector and lens.



5. Lock removed too soon.
Action: Place back lock or replace injector.

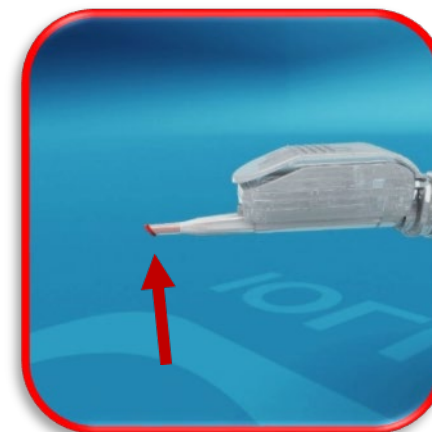

6. Mushroom effect.
Action: Don't push plunger too far.